


SMI-SPON

ABSORBABLE GELATIN SPONGE

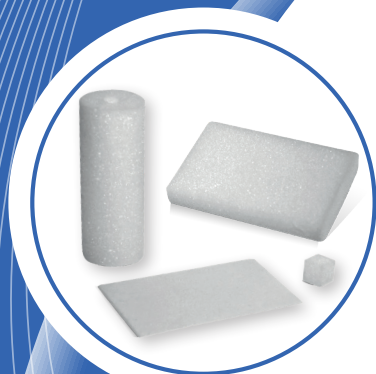


Product highlights

- ✓ Secure and quick haemostasis, stops bleeding effectively - Significantly reduces blood loss
- ✓ Absorbs 30-40 times its own weight in blood and fluids
- ✓ Can easily be reduced in size to any required format- does not stick to gloves or instruments
- ✓ May be applied dry or be soaked with sterile physiological solution or antibiotics
- ✓ Gamma sterilized, individually blister packed and ready to use

Benefits

- ✓ No risk of encapsulation
- ✓ Does not disintegrate after moistening
- ✓ Fasten recovery time for the patient
- ✓ Biocompatible and non pyrogenic, non-toxic, non-allergenic, non-immunogenic
- ✓ Totally absorbed in 3-4 weeks, leaves no residue and encapsulation

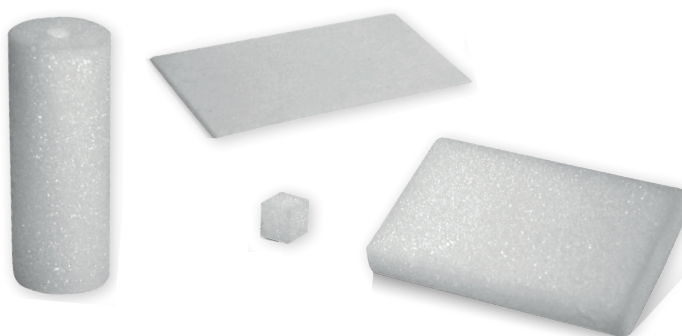

*For quality
&
safety*

smi
www.sutures.be

SMI-SPON

ABSORBABLE GELATIN SPONGE

What is SMI-SPON

SMI-SPON is a surgical haemostatic sponge, manufactured from highly purified first grade gelatin material for use in various surgical procedures.

It is a quick and effective haemostatic which stops bleeding fast. It reduces intraoperative bleeding and surgery time.

Why using SMI-SPON

SMI-SPON can be used in various surgeries for haemostasis. When applied in vivo and used in appropriate amounts, it is completely absorbed within 3-4 weeks. SMI-SPON is biocompatible, non-toxic and non-pyrogenic.

SMI-SPON is gamma sterilized and individually packed in ready to use blister packaging.

Where to use

- ☒ Dental surgery
- ☒ Plastic surgery
- ☒ Vascular surgery
- ☒ Orthopaedic surgery
- ☒ ENT surgery
- ☒ Neuro surgery
- ☒ Abdominal surgery
- ☒ Anorectal surgery
- ☒ Genito Urinary surgery
- ☒ Gynaecological surgery

How to use SMI-SPON

- ☒ Cut a piece to the desired size
- ☒ Apply SMI-SPON either in a dry state or saturated with physiological salt solution
- ☒ If used dry, a piece of SMI-SPON is applied to the bleeding place and held in place with moderate pressure until hemostasis results
- ☒ If used wet, first immerse in the solution and then withdraw, squeeze between gloved fingers to expel air bubbles, and the replace in saline until needed

Product		Unit	Order no.
Regular	70 x 50 x 10 mm	10 Pc.	ZHG705010
Standard	80 x 50 x 10 mm	10 Pc.	ZHG805010
Dental	10 x 10 x 10 mm	32 Pc.	ZHG101010
Tampon	80 x 30 mm DIA	6 Pc.	ZHG8030
Special	80 x 50 x 1 mm	20 Pc.	ZHG805001
Film	200 x 70 x 0.5 mm	10 Pc.	ZHG20070005



*For quality
&
safety*

smi
www.sutures.be