

# SMI TABLET INTRODUCER



The kind way for the administering of tablets to cats and dogs

## Technical attributes

Transparent barrel und blue plunger  
Dimension: 15 cm closed - 18,5 cm open

## Highlights

Smart design with soft silicon tip protects pets mouth  
Easy and fast to use  
Keep your fingers safe with finger bars  
No more wasted pills  
Reusable

## Packing


The box can be converted into a smart display supplied with 10 units



**For quality  
&  
safety**

**smi**  
www.sutures.be

## How to use

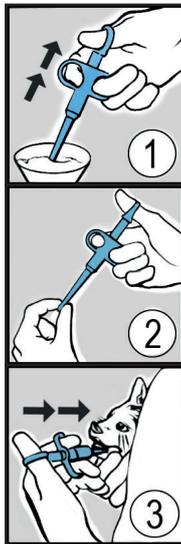

Draw drinking water into the introducer

Place tablet in the holder

Insert tablet introducer and by pressing the piston the tablet is automatically swallowed by the patient

## SMI Tip

Deliver a pill with a little water (optional) to make sure it is swallowed first time

## Cleaning

The tablet introducer is rinsed in tepid water. Avoid using disinfectant detergent.



Product	Unit	Order no
Tablet Introducer with silicone top	10	ZT1